

Description


- The self adhesive incision drape made of high permeable water born polyethylene film, coated with special acrylic adhesive and then attached a layer of protection releasing paper .
- Side bar design make it is easier to paste the dressing on skin or remove.
- Hypoallergenic , will not cause any un-expect action to skin after applied.
- The iodine surgical film contains 0.45%-0.8% iodine , the rate of bacteria killed can reach to 99%.

Instruction of Usage

- Wear the surgical site with sterile drapes and then choose the proper size of sterile surgical dressing film.
- Spread film flat, peel off the protection paper , paste the film on the surgical incision site.
- Smoothing the film from the middle to all surrounding by gauze or by hand, make the film attached on skin closely, cannot left any bubbles in-between the film and skin.
- After the surgery, peeling off the film on both sides and then suturing.

Warning

- Keep dry
- Use only once
- Don't use when package has been opened or damaged
- STERILE, EO

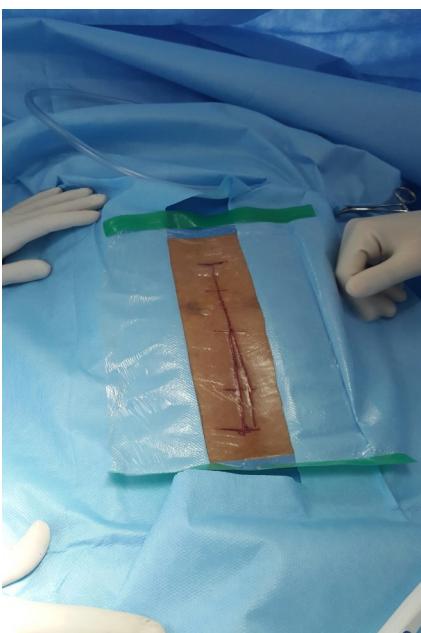

توضیحات

- پوشش محل برش جراحی از پلی اتیلن مقاوم در برابر نفوذپذیری آب تولید شده، که با چسب اکریلیک مخصوص پوشش داده شده است و سپس با یک لایه کاغذ محافظ نچسب تکمیل میگردد.
- لبه های کناری این پوشش چسباندن و جدا کردن آن را از روی پوست موضع جراحی راحت تر مینماید.
- غیر حساسیت ز، پوست پس از برداشتن پوشش هیچگونه واکنشی نشان نخواهد داد.
- فیلم بنادرینه جراحی حاوی ۰.۴۵٪ تا ۰.۸٪ ید میباشد، این مقدار میتواند تا ۰.۹۹٪ باکتریها را از بین ببرد.

روش استفاده

- موقع جراحی را بوسیله شانهای (پوشش های) استریل می پوشانیم و سپس سایز مناسب از فیلم پوشش دهنده استریل را انتخاب می نماییم.
- فیلم را باز نموده کاغذ محافظ آن را جدا می کنیم و فیلم را روی محل برش قرار میدهیم.
- فیلم را از وسط به اطراف صاف نموده بوسیله گاز و یا دست روی آن میکنیم تا کامل به سطح پوست بچسبد و هیچ حبابی بین پوست و فیلم باقی نماند.
- پس از پایان جراحی فیلم را از دوطرف جرا کرده و سپس پوست را بخیه میزنیم.

هشدار

- در جای خشک نگهداری شود
- فقط یکبار مصرف گردد
- زمانیکه بسته بندی باز و یا خراب شده استفاده نشود
- استریل بوسیله اتیلن اکساید

Code No	Model	Size (cm)	Package
9132	Simple	30x30	50Pcs / Box
9134	Simple	30x40	50Pcs / Box
9136	Simple	40x45	50Pcs / Box
9138	Simple	40x60	50Pcs / Box
9144	Iodine	30x40	50Pcs / Box
9148	Iodine	40x60	50Pcs / Box