**NEW
VERSION**

NOCOSPRAY

BIO

DISINFECTION

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Australian Government
Department of Health and Ageing
Therapeutic Goods Administration

ARTG Certificate

Issued to
Aever Young Pty Ltd T/a Equimed
for approval to supply
Aever Young Pty Ltd T/a Equimed - Nocospray - Disinfectant, hospital grade

ARTG Identifier
ARTG Item desc
Product Category
OSD/OT
OSD/OT Description
Intended Purpose
Manufacturer's Details
ARTG

ARTG Standard Conditions:
The above Other Therapeutic Good Listed has been entered on the Register subject to the following conditions:
- Conditions applicable to all therapeutic goods as specified in the document "Standard Conditions Applying to Registered or Listed Therapeutic Goods Under Section 28 of the Therapeutic Goods Act 1989" effective 1 July 1995.
- Conditions applicable to the reference company and class of therapeutic goods as specified in the document "Standard Conditions Applying to Registered or Listed Therapeutic Goods Under Section 28 of the Therapeutic Goods Act 1989" effective 1 July 1995.
Products covered by this Entry:
1. Nocospray - Disinfectant, hospital grade
Product Specific Conditions:
No specific conditions have been recorded against this entry.
Product Standard Indications:
No standard indications have been recorded against this entry.
Product Specific Indications:
No specific indications have been recorded against this entry.

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Result of the sterilisation test carried out during testing of the NocoSystem disinfection appliance

We hereby certify that all the cultures incubated using specimens contaminated with *Mycobacterium tuberculosis*, exposed on work benches and inside level II biosafe environments (in operation) during the testing of the NocoSystem sterilisation apparatus posted a **NEGATIVE** result.

In witness thereof.

(Dr. Enrico Tortoli)

Florence 27 November 2008

Institut Pasteur de Lille
Hospital Hygiene Assessment Service

IPL Report N° : 151003

TRIAL REPORT

Determination of the bactericidal activity of an airborne process of surface disinfection by the NOCOLYSE/NOCOLYSE duo in compliance with a methodology inspired by the NF T 72 281 standard vis-à-vis serotype 1 *Legionella pneumophila*

This trial report concerns only the product (referenced hereunder) submitted to the test.

The methodology was inspired by the NF T 72 281 standard, taking into account the specificities of *Legionella* which can only be cultivated in charcoal-derived environments (opaque agar).

Principle

The test was conducted according to protocol fixed by NF T 72 281 French standard "Airborne processes of surface disinfection".
Determination of bactericidal activity (September 1986) vis-à-vis serotype 1 *Legionella pneumophila*.

Identification of the sample

Name and description of the product: NOCOLYSE (lot 05006031) and the NOCOSPRAY nebulizer
Manufacturer: OXYPHARM - 917, rue Marcel Paul - ZA des Grands Godets - 94508 CHAMPIGNY/MARNE
Date of reception to laboratory: 04/08/2003
Storage conditions: protected from light
Trial period: from 12/09 to 24/09/2003

Experimental conditions

Procedure: sprayed in a directed dispersal
Product: NOCOLYSE (lot 05006031)
- composition: Hydrogen peroxide, catalyst, biosurfactant and excipient
- spray time: 3'50
- quantity used: 64 ml
NOCOSPRAY unit: Timer set in proportion to the volume of the room to cover (graduated from 25 to 25/m3)

Institut Pasteur de Lille
Service of experience to hygiene hospitalization

THE ACTUAL PROGRESS OF THE PRODUCT NOCOLYSE AGAINST VIRAL ACTIVITY

According to the norm of NF T 72 180

Report IPL no.: 161103

This report is specially for the product mentioned under trial only.

Material and Methods:
Identification of samples
Name and description of product : NOCOLYSE .
Compendary: Oxypharm - 917, rue Marcel Paul - ZA des Grands Godets - 94508 CHAMPIGNY SUR MARNE Cédex .
Date of reception in the laboratory : 04/08/2003
PH : 5
Aspect of product : uncolored.

1) Material :

a) **Product used :**
The product Nocolyse used in 1/10⁶

b) **Medium used :**
Medium used for the cellular culture
- Cells VERO ATCC 81 CCL : EAGLE MEM medium + amino acids+ serum of calf fetus + glutamine+ gentalline 8µg/ml.
- Cells KB : EAGLE MEME medium + serum of calf fetus+ glutamine + gentalline 8µg/ml.

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